

CLEANING

We recommend keeping the surface of the silicone gel clean and clear of debris. Gently clean both sides of the Sleeve and Cuff with soap and water. After cleaning, hang the Sleeve up to dry.

WEEKLY DISINFECTION

If the Sleeve is worn so that the gel portion of the seal touches the skin, then the area of the gel that touches the skin should be disinfected once a week.

1. Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth.
2. Lightly wipe the gel side of the sleeve with the cloth.
3. Rinse the alcohol from the sleeve with water.
4. Hang the sleeve up to dry.

Note: Do not soak the Sleeve in the ethyl or isopropyl alcohol. Extended contact with large amounts of ethyl or isopropyl alcohol will stiffen the Sleeve.

STORAGE

When the prosthesis is not being worn, the bottom portion of the Sleeve should be left on the socket. Do NOT leave the Sleeve on the socket with the top half of the Sleeve rolled back onto the socket, or the Sleeve will stretch out. If the prosthesis is not to be worn for five days or more, the Sleeve should be removed, cleaned, and stored in a clean, dry place.

FIXING A LEAK IN THE SLEEVE

Accidental impacts may occasionally perforate the LimbLogic Sleeve. These perforations may then prevent optimal sealing of the Sleeve and LimbLogic System.

You may be able to fix a leak simply by separating the Sleeve from the inner Cuff, rotating the Cuff slightly on the socket, then putting the Sleeve back on over the Cuff (refer to Steps 3-6 on the opposite page). This will misalign any holes in either piece and allow the Sleeve to seal up again.

If this does not eliminate the leak, contact your prosthetist.

WillowWood™

LimbLogic® Sleeve Patient Instructions



Please contact your prosthetist if you have any questions about the use of the LimbLogic Sleeve and Cuff.

WillowWood™

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com




Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands




PN-2270-A 04 OCT 2012

PUTTING ON THE SLEEVE AND CUFF


1. Turn the Sleeve inside out so that the fabric side is facing in.
2. Pull the Sleeve over the socket, with the large end of the Sleeve first. Continue pulling the Sleeve until it is completely past the socket onto the prosthesis.

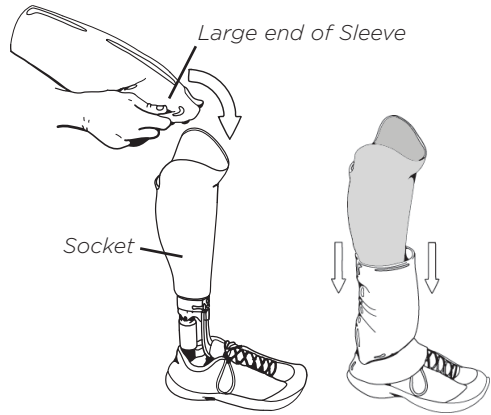


3. Apply the Cuff so that the small end of the Cuff is on the socket, and the large end of the Cuff extends approximately 3 to 5 cm above the highest edge of the socket.

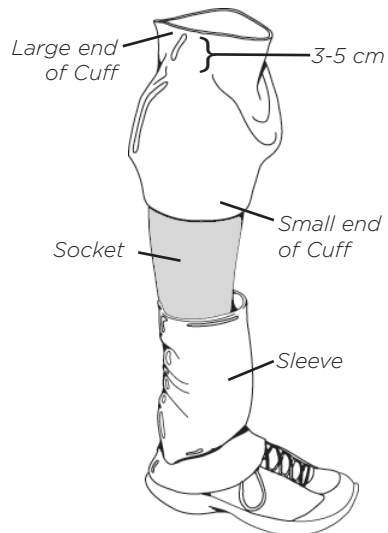


4. Fold the large end of the Cuff down over the socket to expose the open end of the socket. Make sure that none of the Cuff extends higher than the socket at any point.

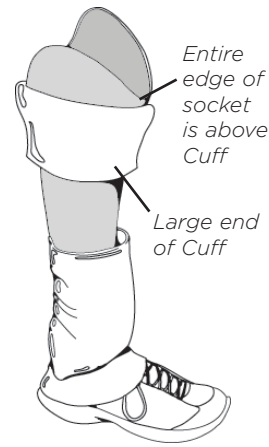


5. Step into the socket and pull the large end of the Cuff up onto the liner on the residual limb. Make sure there are no wrinkles or creases in the Cuff, and make sure the Cuff does not get pinched between the liner and the socket.

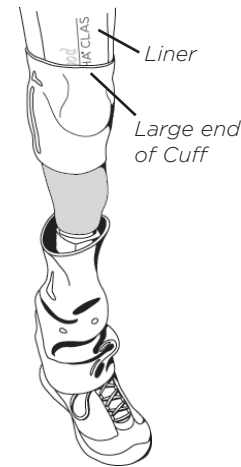

6. Roll the Sleeve up over the Cuff, sealing the Sleeve to the Cuff while also sealing the top of the Sleeve to the liner.

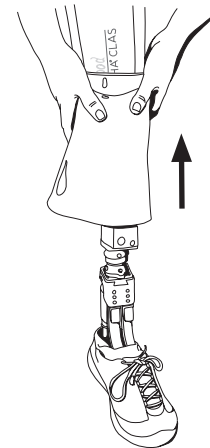

CORRECT/INCORRECT POSITIONING OF SLEEVE AND CUFF

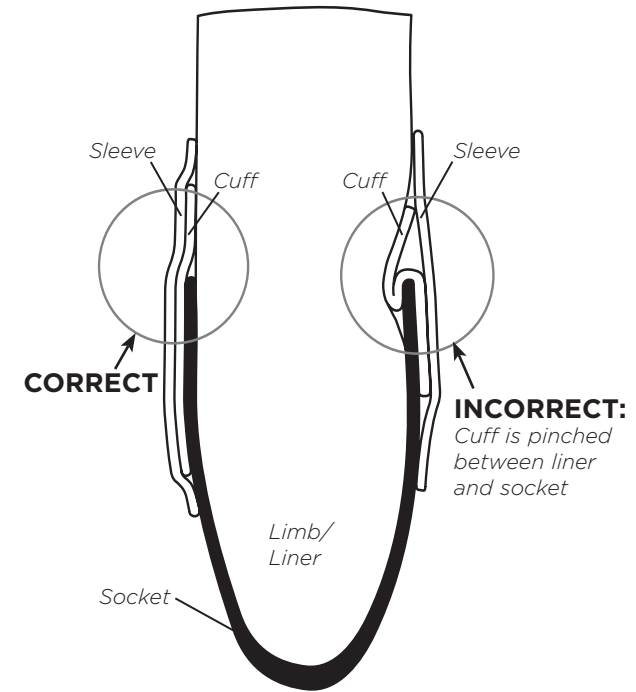

After you wear the paired Sleeve/Cuff the first time, the two should stick together. The next time you put on the prosthesis, you should be able to roll up the Sleeve and the Cuff as one piece.

If desired, you may occasionally reposition the Cuff (move it up or down, or rotate it) with respect to the Sleeve to help extend the life of the products.

If, over time, the Sleeve and Cuff stop sticking together, simply separate them, clean them thoroughly with soap and water, and put them back on again as shown here.