

# LimbLogic® Quick-Start Guide

## 1. Turn on the Vacuum Pump.

Press one time.  
Pump will beep once.



Wait 10 seconds.

## 2. Turn on the fob.



Press and hold until digits flash in "chasing" pattern. Fob displays current vacuum level. Pump will operate until preselected vacuum level has been reached.

## 3. To enter Standby Mode:



Press for approximately 2 seconds. Chasing pattern will be displayed, then Standby Indicator will light.



## To resume vacuum regulation:



Press for approximately 2 seconds. Chasing pattern will be displayed as pump returns to Active Mode.

## 4. Turn off the Vacuum Pump.

Press one time.  
Pump will beep twice.



# Fob Guide


## Vacuum Pump Battery Indicator

 50-100%

 30-50%

 20-30%

 < 20%

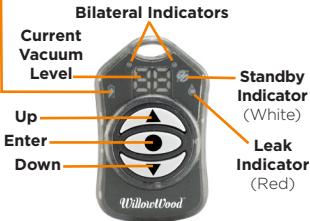

Refer to full manual for complete instructions.

## *WillowWood*

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 [www.willowwoodco.com](http://www.willowwoodco.com)



Ohio Willow Wood Company B.V  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands

PN-2015-F  
22 JAN 2013